



N95 AD5N95

USER INSTRUCTIONS

N95 RESPIRATOR

This respirator provides a minimum 95% efficiency against certain non-oil particulate aerosols. Further information can be obtained from Dentec Safety Specialists Corp. Toll Free at 1-888-533-6832.

WARNINGS:

1. Wearer must read fitting instructions and use limitations before use.
2. The respirator does not protect against the risk of contracting disease or infection.
3. Before occupational use of this respirator a written respiratory protection program must be implemented meeting all the local government requirements. In the United States employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing.

FITTING INSTRUCTIONS



1. Hold the respirator in hand with the nosepiece at your fingertips, allowing the headbands to hang freely below your hand.



2. Press the respirator firmly against your face with the nosepiece on the bridge of your nose.



3. Stretch and position the top band high on the back of the head. Stretch the bottom band over the head and position below your ears.



4. Using both hands, mold nosepiece to the shape of your nose.



5. To test fit:
 - a) Cup both hands over the respirator being careful not to disturb position, and
 - b) inhale vigorously. If air leaks around the edges, reposition the straps or adjust strap tension for a better fit.

Caution: The respirator should be fit checked prior to each wearing.



Foldable cup shape, easy to carry.



See Matrix For
Specific Approvals

AD5N95B20



Non-medical use



www.dentecsafety.com